



The next level of deliverability<sup>1</sup>



Ultrathin struts<sup>2</sup>



Outstanding patient outcomes<sup>3</sup>




Technical data / ordering info

Vascular Intervention // **Coronary**  
Drug-Eluting Stent System



# Orsiro<sup>®</sup> Mission<sub>DES</sub>

Even better deliverability for the outstanding Orsiro DES

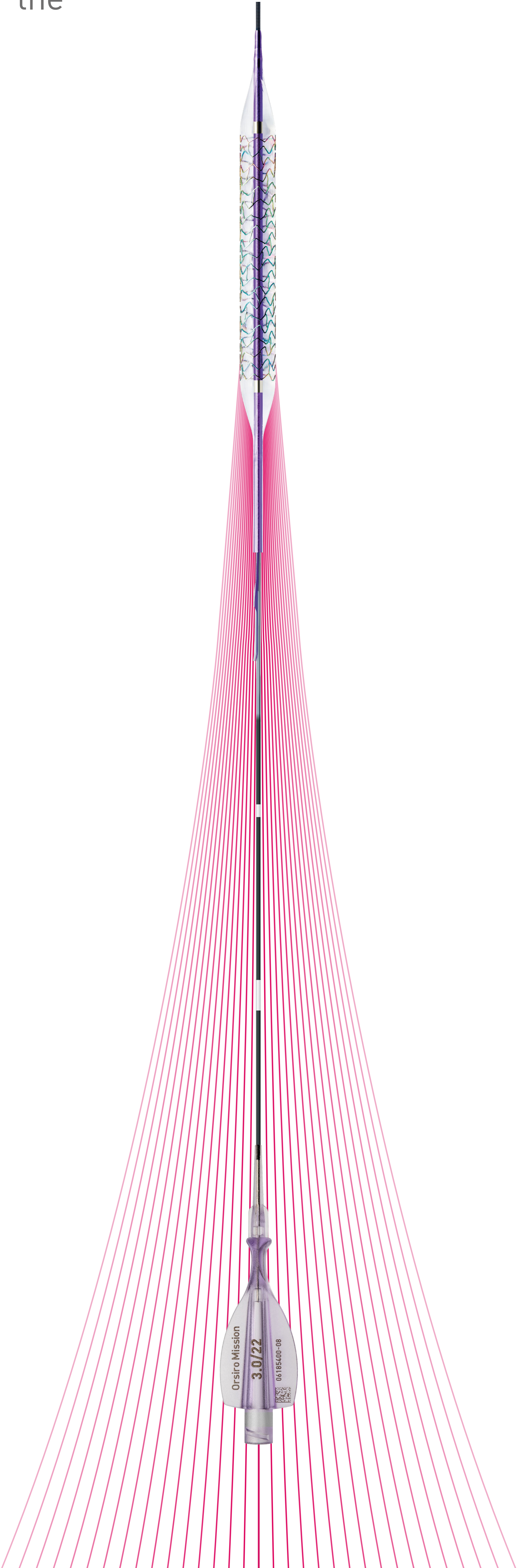






# Orsiro Mission<sup>DES</sup>

Even better deliverability for the outstanding Orsiro DES

## The next level of deliverability<sup>1</sup>

### 1st in Push<sup>4</sup>

Transmitting up to **96% more** force from hub to tip.

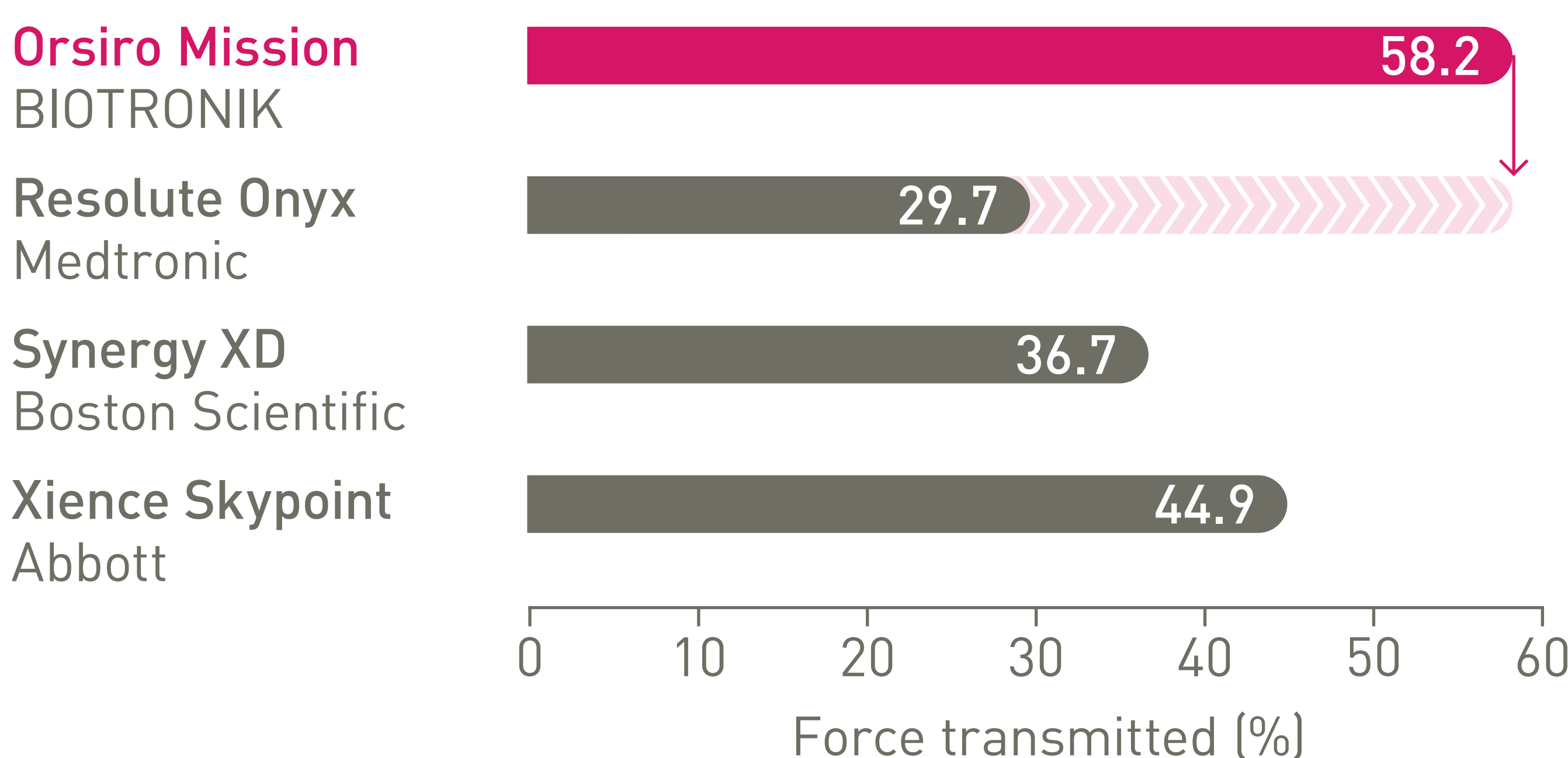


### 1st in Track<sup>4</sup>

Up to **33% less** force needed to follow the path to the lesion.




### 1st in Cross<sup>4</sup>

Up to **64% less** force needed to successfully cross demanding anatomies.

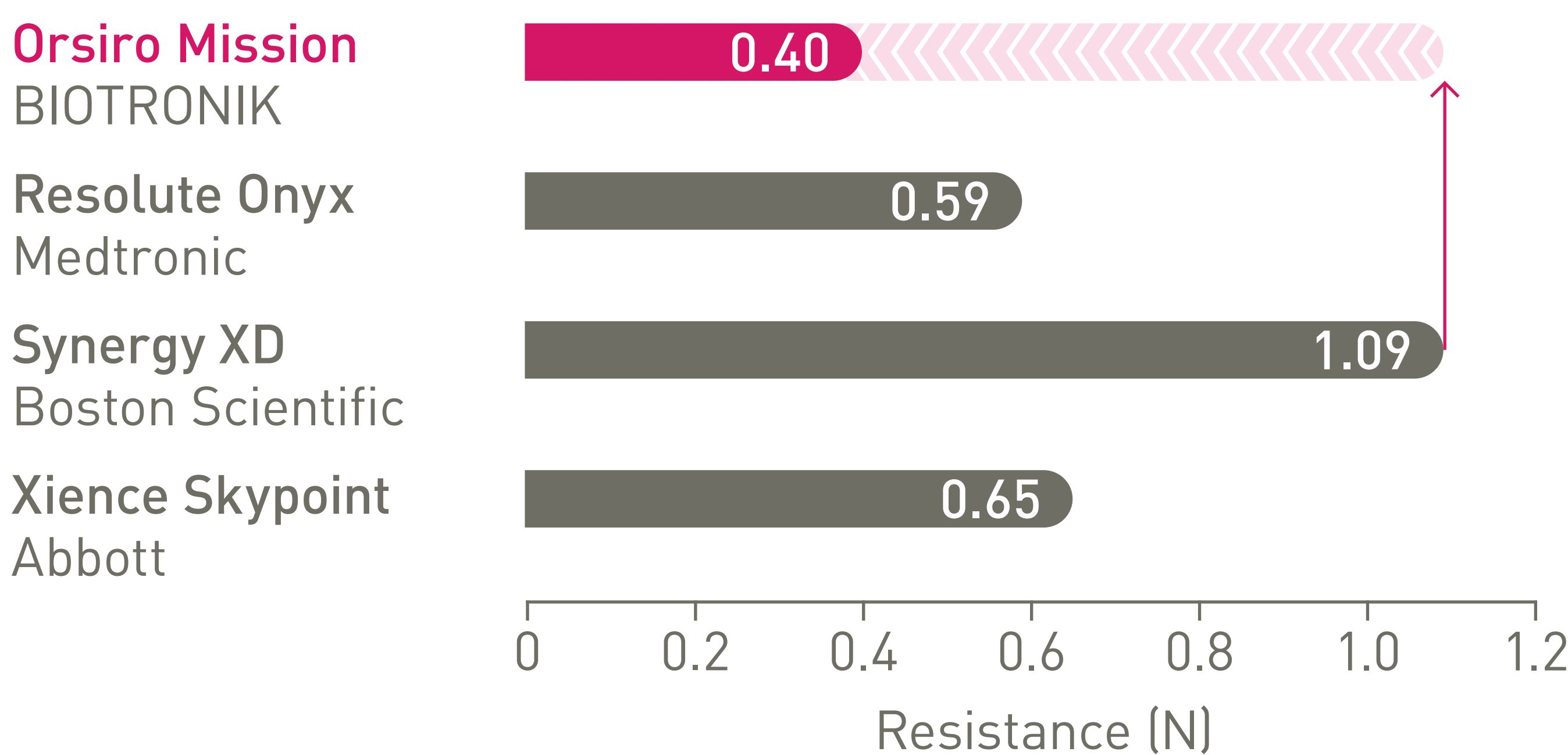

Proven deliverability on the bench and in a **real-world** user evaluation of over 1,000 implantations<sup>5</sup>:

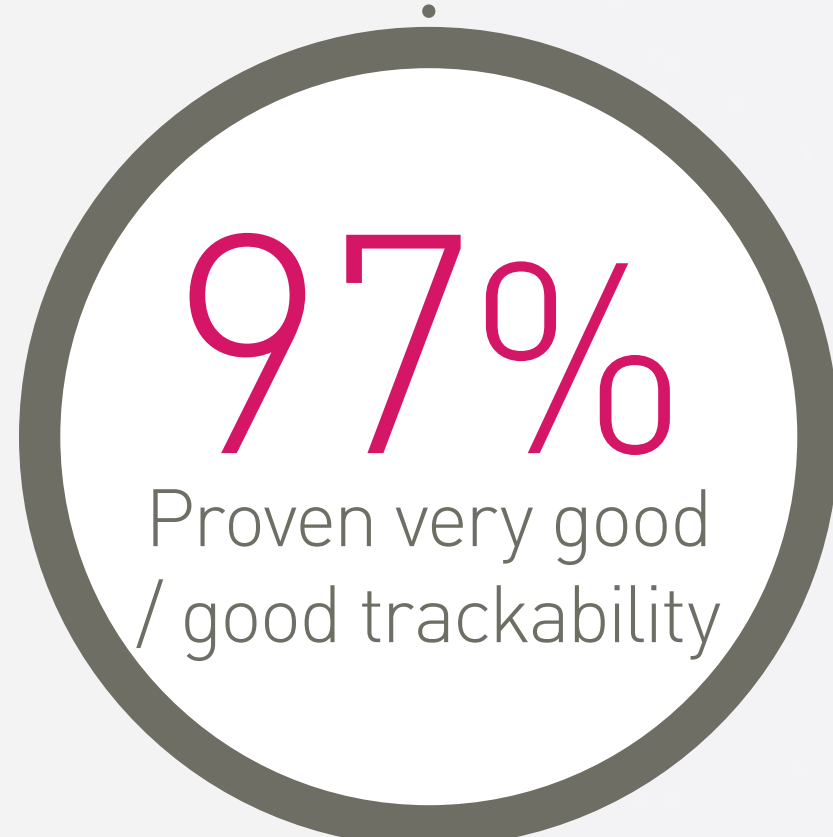


“Lesion crossing with low friction, reliable performance”

Dr. Mathias Brandt,  
Paracelsus Medical University,  
Salzburg, Austria





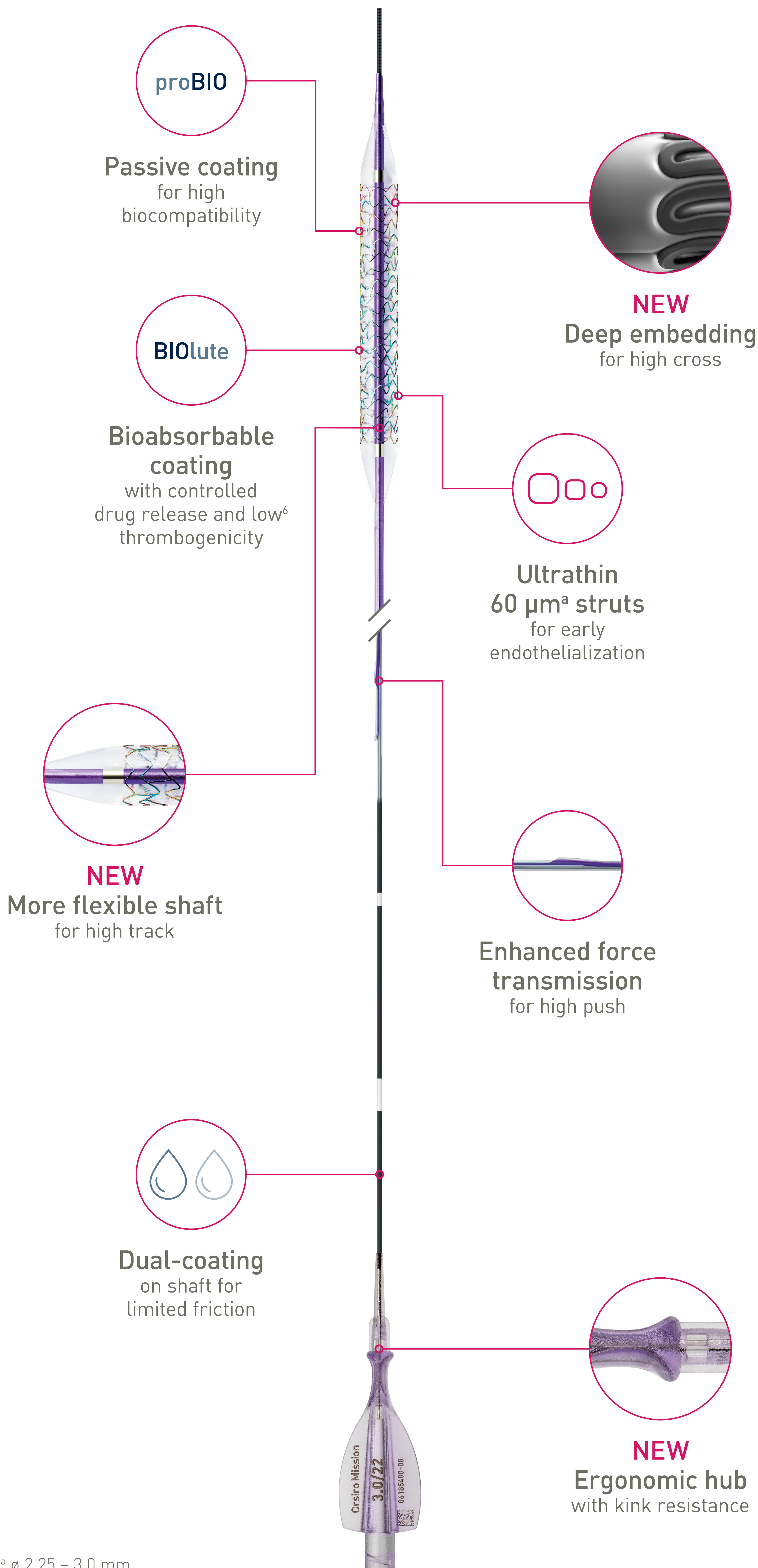



# Ultrathin struts<sup>2</sup>

## For early endothelialization


Strut thickness  
in perspective<sup>7</sup>

**Orsiro**  
BIOTRONIK  
CoCr-SES



60 µm<sup>a</sup>

**Synergy XD**  
Boston Scientific  
PtCr-EES




74 µm

**Ultimaster**  
Terumo  
CoCr-SES




80 µm

**Resolute Onyx<sup>8,9</sup>**  
Medtronic  
CoNi-ZES




81 µm

**Xience Family**  
Abbott  
CoCr-EES




81 µm

**Promus**  
Boston Scientific  
PtCr-EES




81 µm


**BioMatrix**  
Biosensors  
316L-BES

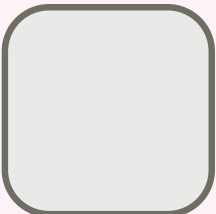


120 µm

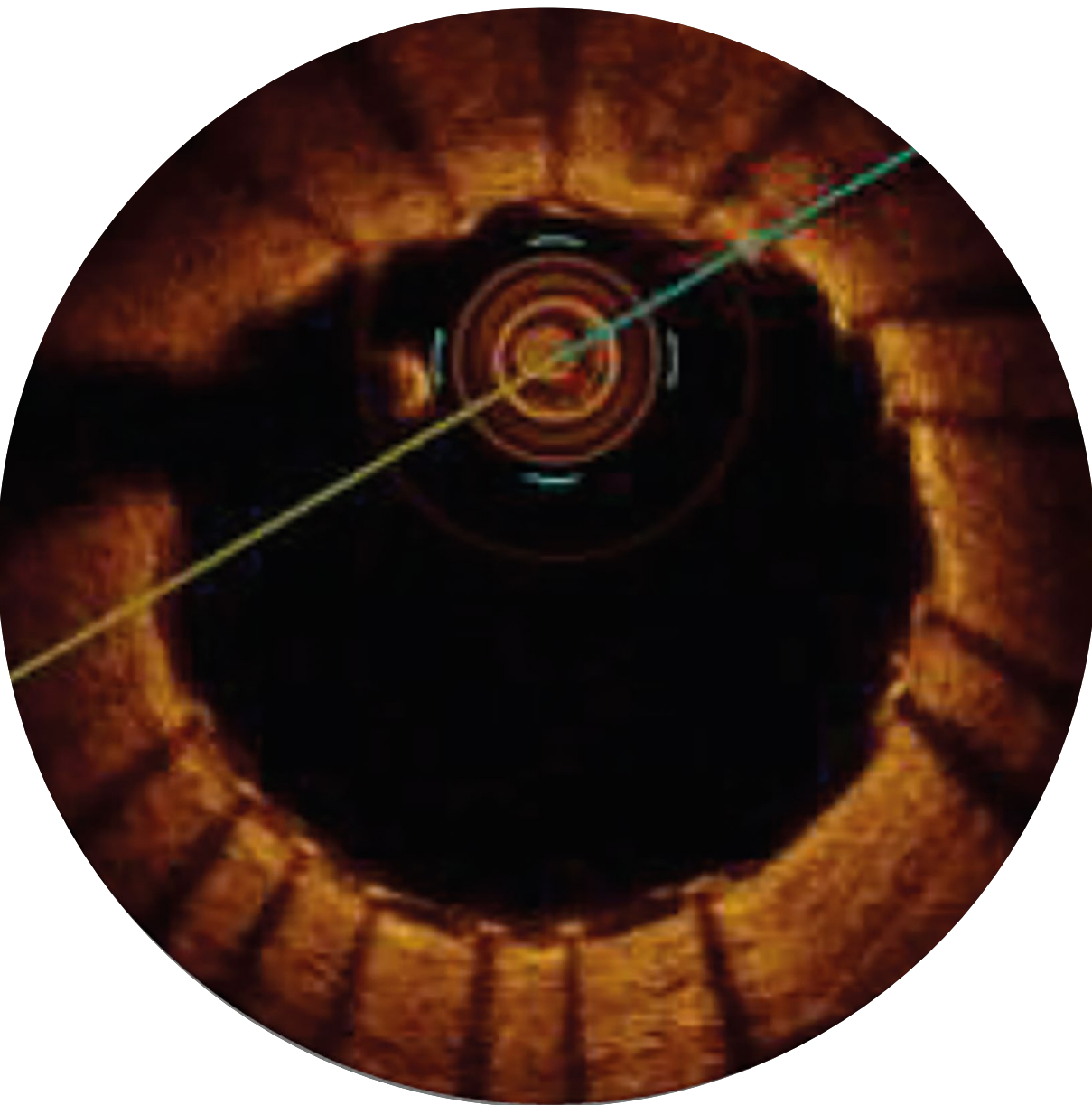

Strut coverage<sup>10</sup>  
**30 days<sup>b</sup>**  
>80%  
n = 589

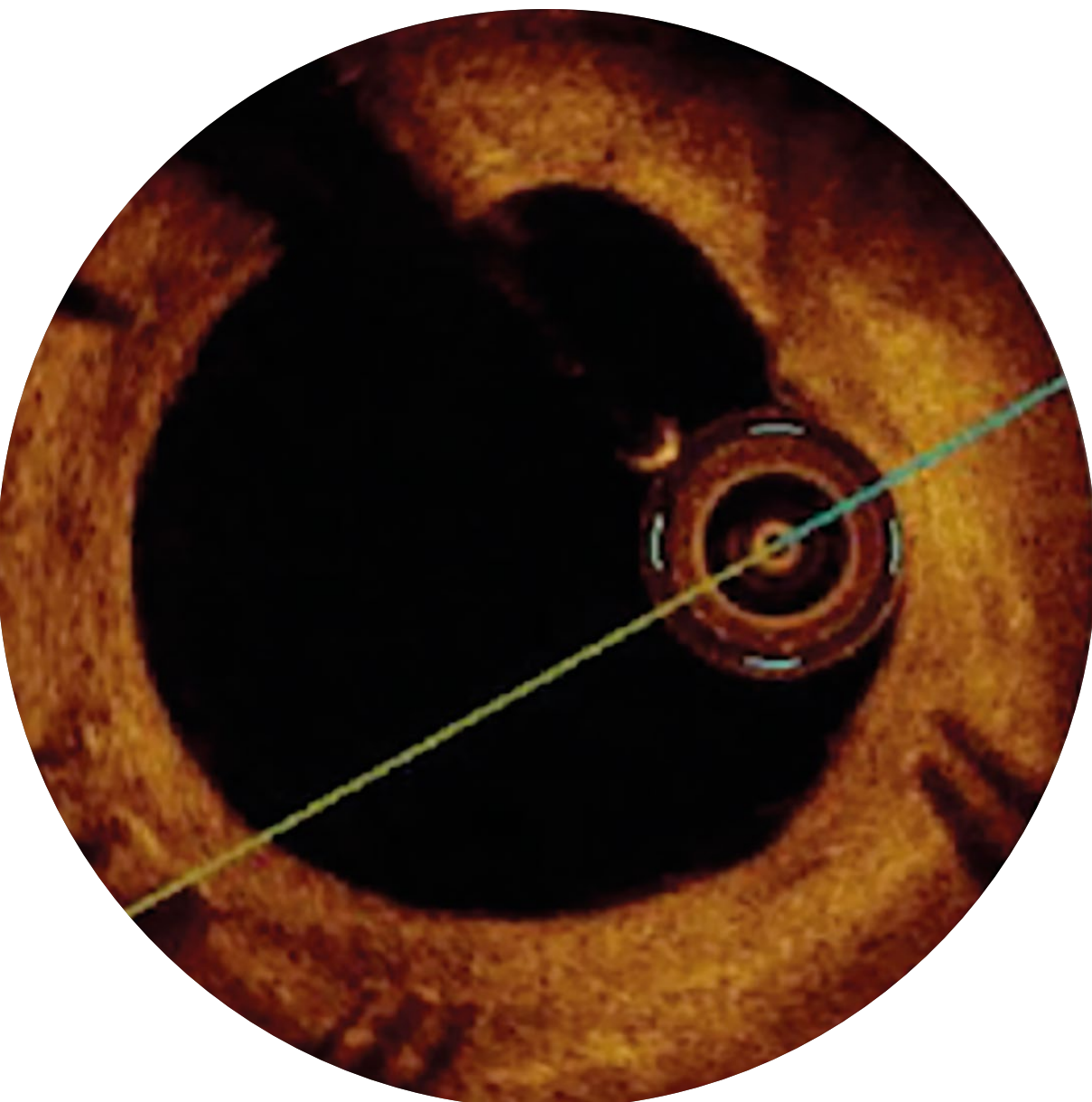

Strut coverage<sup>10</sup>  
**90 days<sup>b</sup>**  
>97%  
n = 874

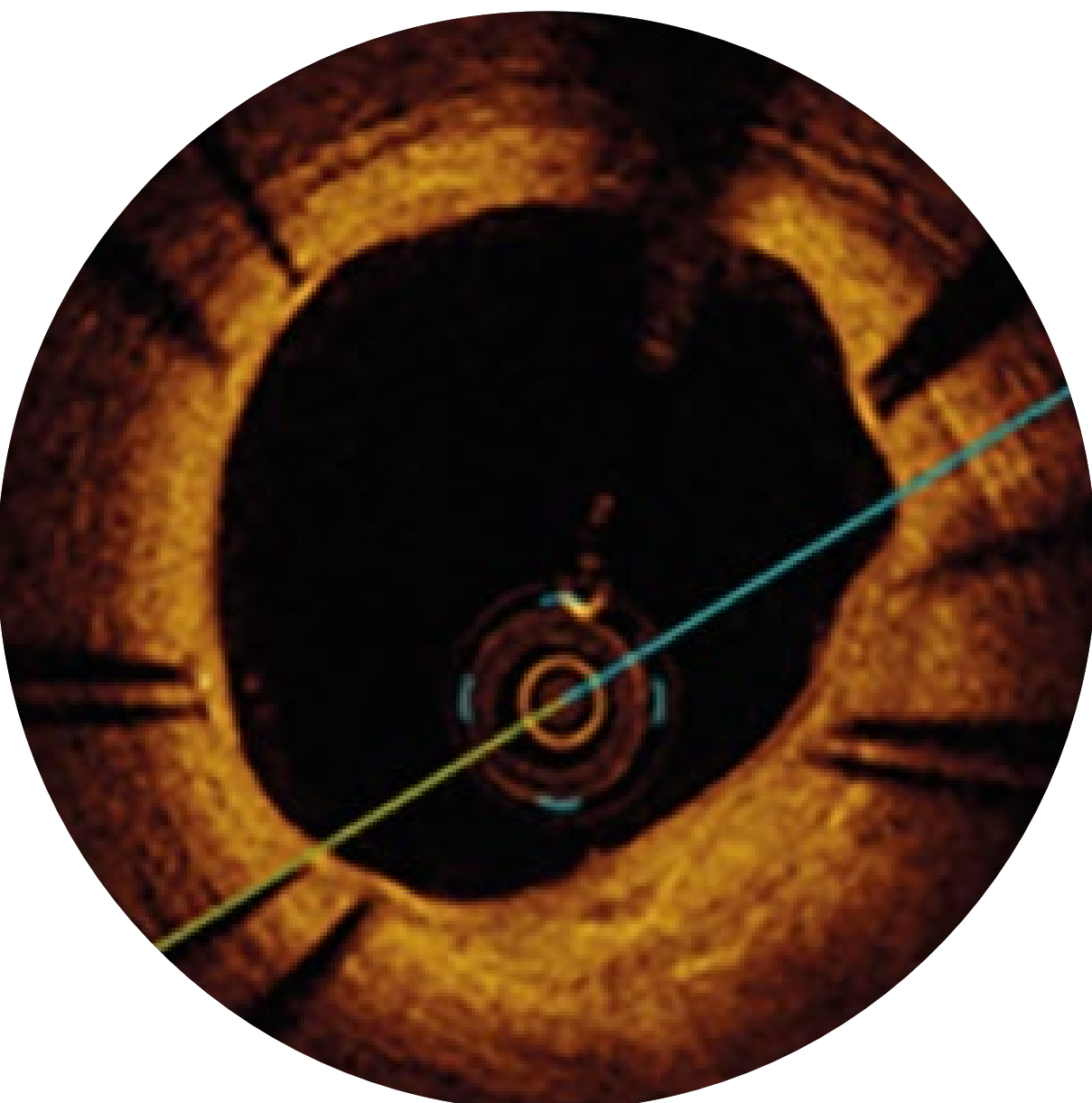

Strut coverage<sup>10</sup>  
**180 days<sup>b</sup>**  
>98%  
n = 1,130



n = number of struts analyzed  
<sup>b</sup> Images: Secco G et al. Time-related changes in neointimal tissue coverage following a new generation SES implantation: an OCT observational study. Presented at: euro PCR, May 20, 2014; Paris, France.

<sup>a</sup> ø 2.25 – 3.0 mm




Outstanding patient outcomes<sup>3</sup>

One of the most studied DES<sup>11</sup>



Orsiro – the highest probability (70.8%) to rank as the best stent<sup>c</sup>

Taglieri et al. network meta-analysis (n = 99,039 patients, 77 RCTs)<sup>13</sup>

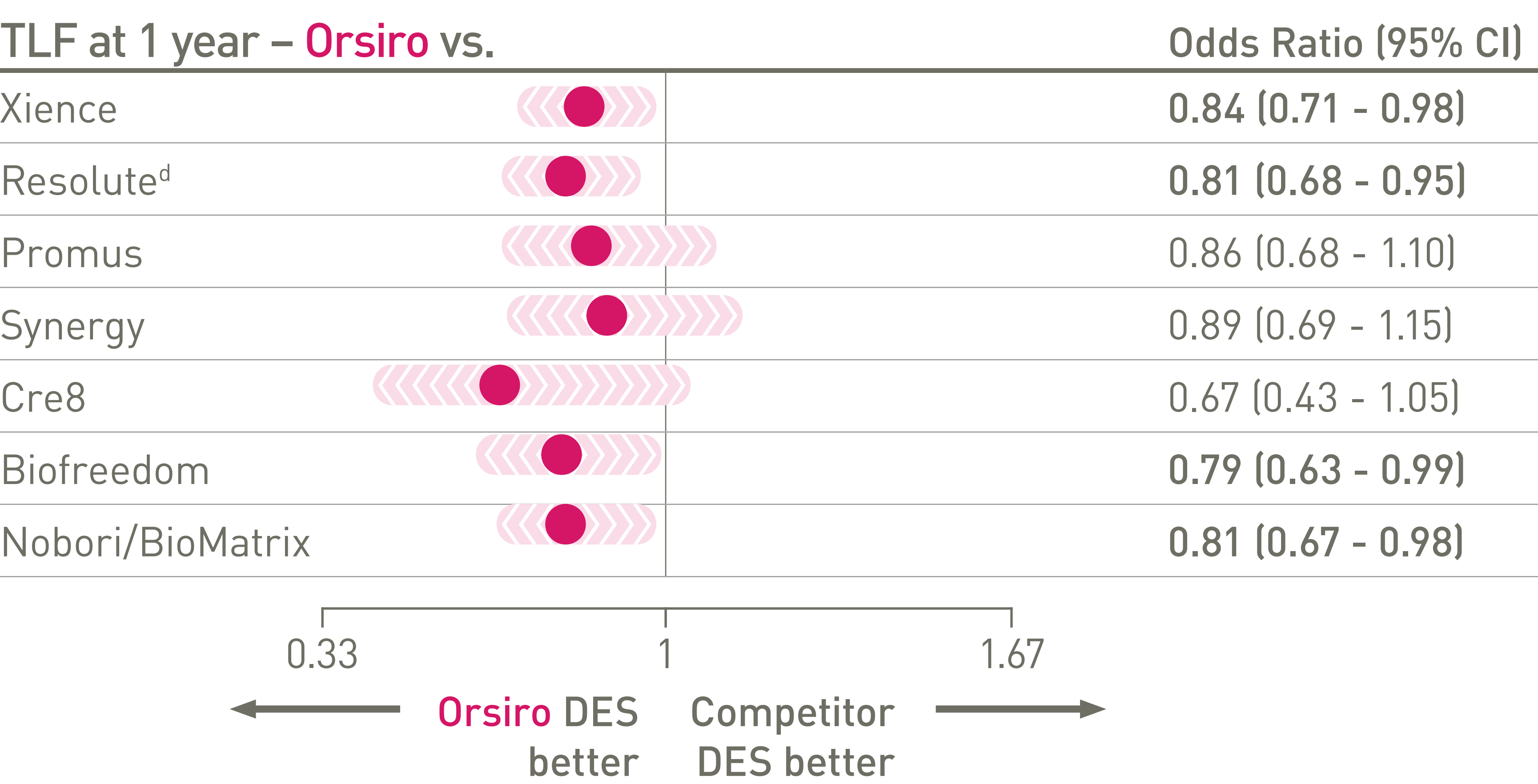


“If we want to inform our clinical practice on the best evidence available, we have to acknowledge that at 1-year the best stent, is the Orsiro stent.”


Dr. Tullio Palmerini,  
Policlinico S. Orsola, Malpighi,  
Bologna, Italy

Pushing the boundaries of safety performance with Orsiro<sup>e</sup>

BIOFLOW-V (n = 1,334), 5-Year results of the FDA pivotal trial<sup>14</sup>

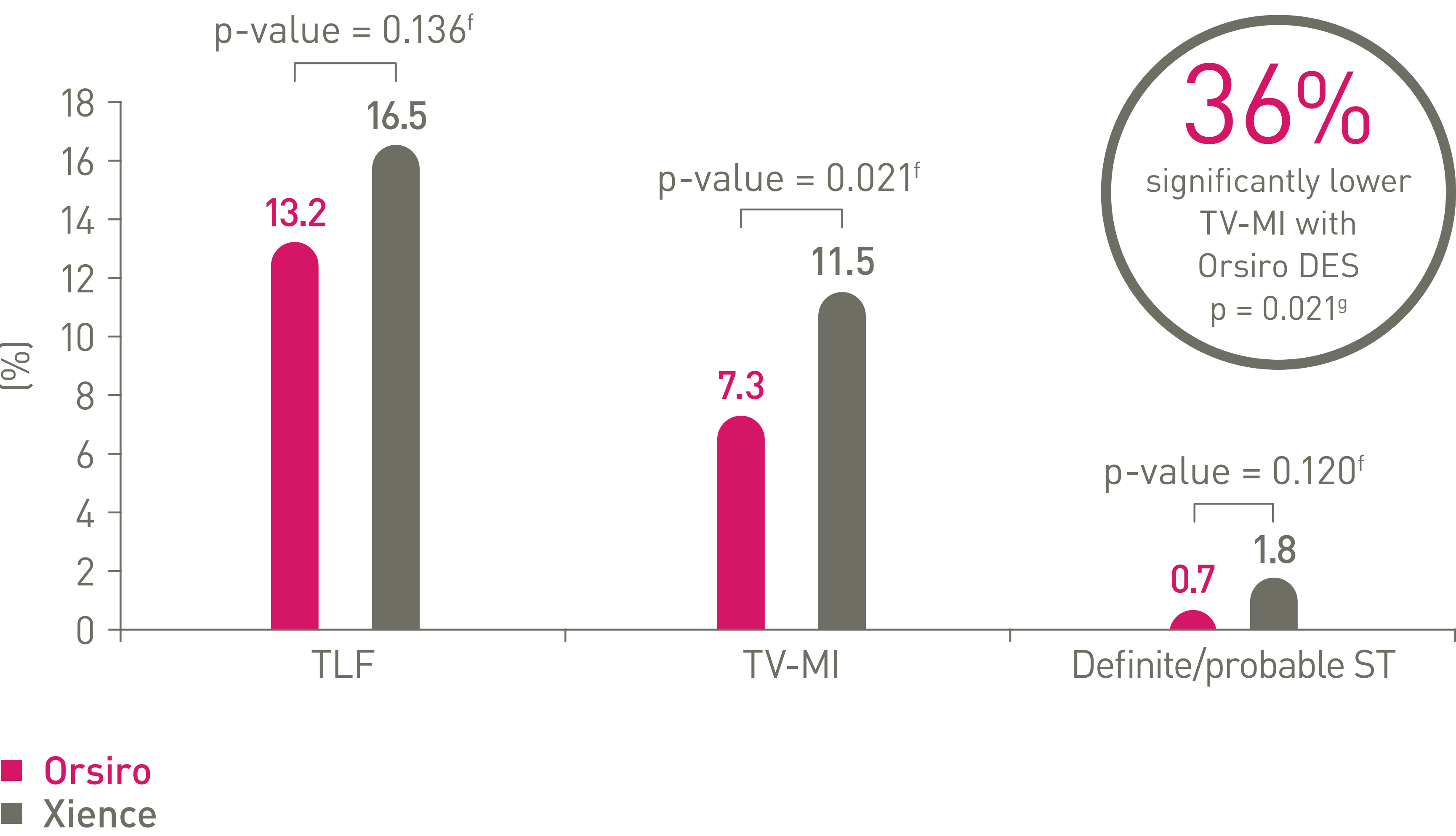

Orsiro Mission DES is indicated for complex patients and lesions<sup>h</sup>

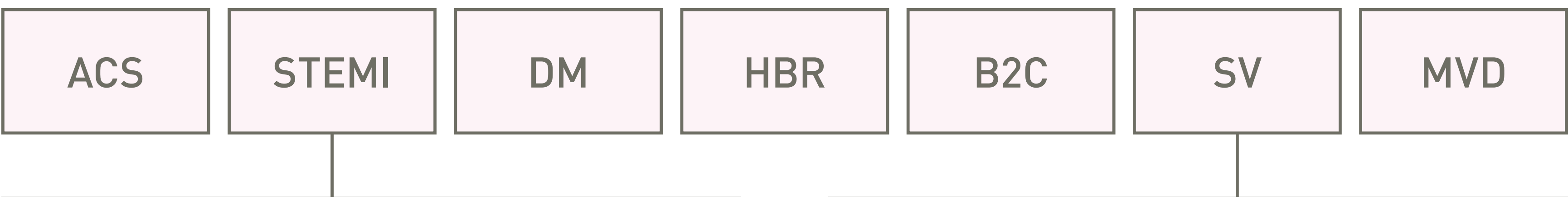


Continued Superiority in STEMI at 2 years<sup>15,i</sup>  
BIOSTEMI (n = 1,300)



Target Lesion Failure (TLF) rate at 2 years.

Rate Ratio (95% BCI<sup>j</sup>): 0.58 (0.40-0.84)  
Posterior probability of Superiority: 99.8%  
Bayesian ITT Population<sup>k</sup>

Low stent thrombosis (ST) at 5 years<sup>16</sup>  
BIO-RESORT Small Vessels (n = 1,506)

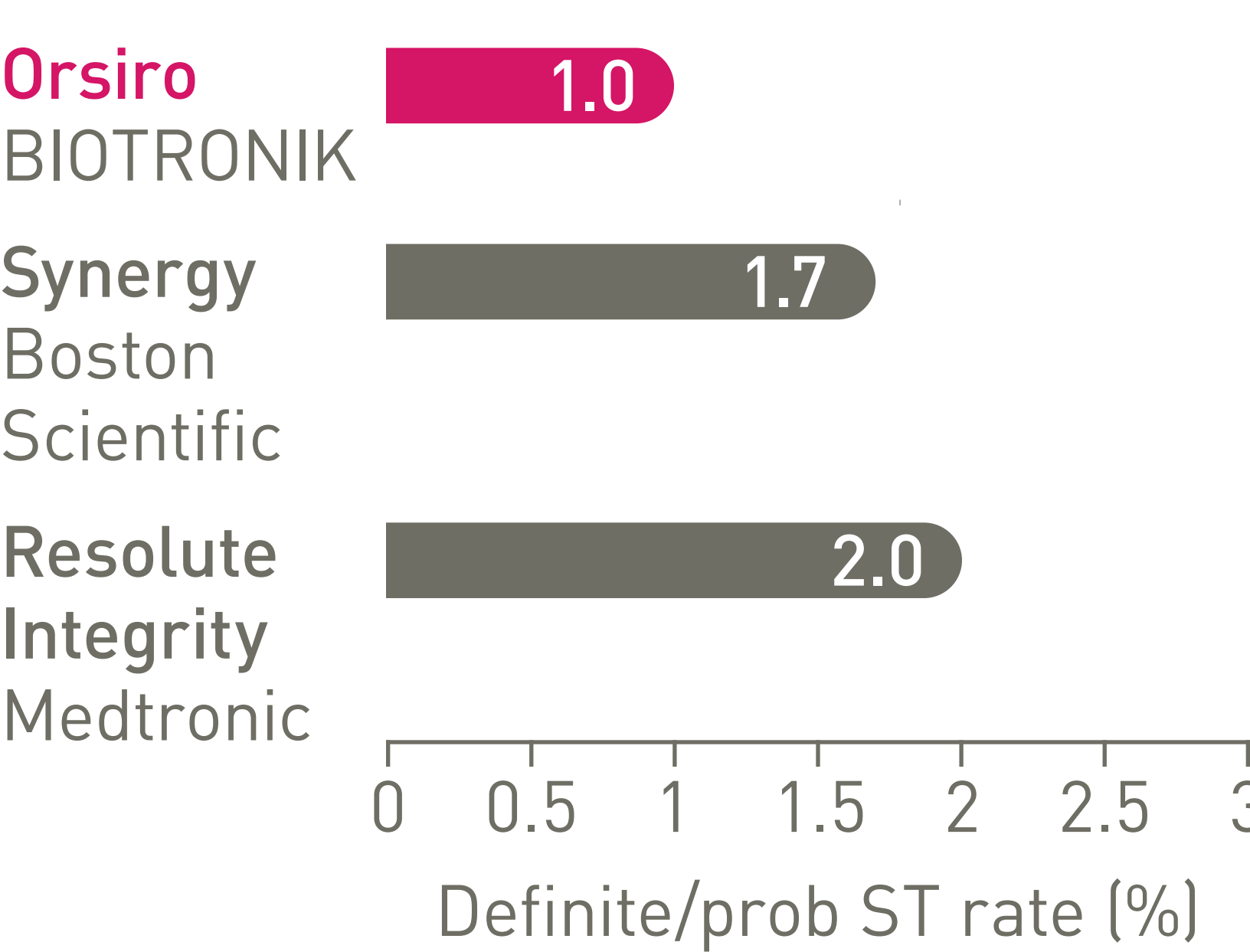

<sup>c</sup> Based on 1-year TLF SUCRA score, in comparison to Xience, Resolute and Nobori/BioMatrix, after a median follow-up period of 50 months; <sup>d</sup> Resolute Integrity and Resolute Onyx; <sup>e</sup> In comparison to Xience, based on statistically significant lower TV-MI and late/very late definite/probable ST rates from the BIOFLOW-V trial through 5 years; <sup>f</sup> p-values for 60-month frequentist analysis; <sup>g</sup> In comparison to Xience, based on BIOFLOW-V 5-year results; <sup>h</sup> As per IFU: ACS – Acute Coronary Syndrome; STEMI – ST-Elevation Myocardial Infarction; DM – Diabetes Mellitus. HBR – High Bleeding Risk; B2C – Complex Lesions; SV – Small Vessels; MVD – Multi-Vessel Disease; <sup>i</sup> In comparison to Xience, based on TLF, in the BIOSTEMI trial at 2 years; <sup>j</sup> BCI: Bayesian Credibility Interval; <sup>k</sup> n= 1,300 newly enrolled STEMI patients including 407 patients from the BIOSCIENCE STEMI subgroup used as prior information; <sup>l</sup> In comparison to Xience, based on a Rate Ratio of 0.58, in the BIOSTEMI trial at 2 years; <sup>m</sup> In comparison to Resolute Integrity, based on 5-year results of the BIO-Resort trial SV subgroup.



| Indication | Orsiro Mission is indicated for improving coronary luminal diameter in patients with symptomatic ischemic heart disease due to discrete de-novo stenotic lesions and in-stent restenotic lesions (length ≤ 40 mm) in the native coronary arteries with a reference vessel diameter of 2.25 mm to 4.0 mm including the following patient and lesion subsets: |  |
|------------|---|--|
|            | Acute Coronary Syndrome (ACS)<br>ST-Elevation Myocardial Infarction (STEMI)<br>Diabetes Mellitus (DM)<br>Complex Lesions (B2/C)<br>High Bleeding Risk (HBR)   | Long Lesions (LL) (e.g. ≥ 20 mm)<br>Small Vessels (SV) (e.g. ≤ 2.75 mm)<br>Multi-Vessel Disease (MVD)<br>Male/Female<br>Old Patients (e.g. > 65 y) |

| Technical Data | Stent  |
|----------------|--|
|                | Stent materialCobalt chromium, L-605   |
|                | Strut thicknessø 2.25 – 3.0 mm: 60 µm (0.0024");<br>ø 3.50 – 4.0 mm: 80 µm (0.0031")                                   |
|                | Passive coatingproBIO (Amorphous Silicon Carbide)  |
|                | Active coatingBIOLute bioabsorbable Poly-L-Lactide (PLLA) eluting a limus drug   |
|                | Drug dose1.4 µg/mm²  |
|                | Delivery system  |
|                | Catheter typeRapid exchange  |
|                | Recommended guide catheter5F (min. I.D. 0.056")  |
|                | Guide wire diameter0.014"  |
|                | Usable catheter length140 cm   |
|                | Balloon materialSemi crystalline polymer material  |
|                | Coating (Distal shaft)Hydrophilic  |
|                | Coating (Proximal shaft)Hydrophobic  |
|                | Marker bandsTwo swaged platinum-iridium markers  |
|                | Lesion entry profile0.017"   |
|                | Distal shaft diameter2.7F: ø 2.25 – 3.0 mm; 2.9F: ø 3.5 – 4.0 mm   |
|                | Proximal shaft diameter2.0F  |
|                | Nominal pressure (NP)10 atm  |
|                | Rated burst pressure (RBP)16 atm   |
|                | Storage  |
|                | Use Before Date (UBD)24 months   |
|                | TemperatureBetween 15°C (59°F) and 25°C (77°F), short term excursions between 10°C (50°F) and 40°C (104°F) are allowed |

| Ordering Information | Stent ø (mm) | Stent Length (mm) |        |        |        |        |        |        |        |        |
|----------------------|--------------|-------------------|--------|--------|--------|--------|--------|--------|--------|--------|
|                      |              | 9                 | 13     | 15     | 18     | 22     | 26     | 30     | 35     | 40     |
|                      | 2.25         | 419101            | 419107 | 419113 | 419119 | 419125 | 419131 | 419137 | 419143 | 419149 |
|                      | 2.5          | 419102            | 419108 | 419114 | 419120 | 419126 | 419132 | 419138 | 419144 | 419150 |
|                      | 2.75         | 419103            | 419109 | 419115 | 419121 | 419127 | 419133 | 419139 | 419145 | 419151 |
|                      | 3.0          | 419104            | 419110 | 419116 | 419122 | 419128 | 419134 | 419140 | 419146 | 419152 |
|                      | 3.5          | 419105            | 419111 | 419117 | 419123 | 419129 | 419135 | 419141 | 419147 | 419153 |
|                      | 4.0          | 419106            | 419112 | 419118 | 419124 | 419130 | 419136 | 419142 | 419148 | 419154 |

1. In comparison to Xience Sierra, Resolute Onyx and Synergy for bench tests on pushability, trackability and crossability, BIOTRONIK data on file; 2. As characterized with respect to strut thickness in Bangalore et al. Meta-analysis; 3. Based on investigator’s interpretation of BIOFLOW-V primary endpoint result; 4. BIOTRONIK data on file; 5. Evaluation of Market Acceptance, BIOTRONIK data on file; 6. Per investigators’ interpretation of preclinical studies with Orsiro as mentioned in Cassese et al. J Thorac Dis 2018;10(2):688-692; 7. Stefanini GG et al. Coronary stents: novel developments. Heart. 2014 Jul 1;100(13):1051–61; 8. Low AF. Stent platform for procedural success: Introducing the Continuous Sinusoidal & Core Wire Technologies. Presented at: AsiaPCR; 22-24 January, 2015; Singapore, Singapore; 9. Tolentino A. Evolving DES Strategy: Biodegradable Polymer vs. Bioabsorbable Scaffold. Presented at: Cardiovascular Nurse/Technologist Symposium; June 17, 2016; New York, USA; 10. Secco G et al. Time-related changes in neointimal tissue coverage of a novel Sirolimus eluting stent: Serial observations with optical coherence tomography. Cardiovascular Revascularization Medicine 17.1 (2016): 38-43; 11. Based on Taglieri et al. Meta-analysis, against currently used DES; 12. BIOTRONIK data on file, as of January 2020; 13. Taglieri N et al. Target lesion failure with current drug-eluting stents: Evidence from a comprehensive network meta-analysis. JACC 2020 13(24):2868-78; 14. Kandzari D et al. Ultrathin Bioresorbable Polymer Sirolimus-Eluting Stents versus Thin Durable Polymer Everolimus-Eluting Stents for Coronary Revascularization: Final 5-year Outcomes from the Randomized BIOFLOW V Trial, Submitted manuscript to JACC, 2022: NCT02389946; 15. Pilgrim et al. Biodegradable – versus durable-polymer drug-eluting stents for STEMI. Final 2-year outcomes of the BIOSTEMI trial. J Am Coll Cardiol. Cardiovasc Interven. 2021, doi: 10.1016/j.jcin.2020.12.011; 16. Ploumen et al. BIO-RESORT Small Vessels 5Y-EuroPCR2022.

Orsiro, Orsiro Mission, proBIO and BIOLute are trademarks or registered trademarks of the BIOTRONIK Group of Companies. Synergy, Synergy XD, Promus and Taxus are trademarks or registered trademarks of the Boston Scientific Group of Companies. Resolute, Integrity, Resolute Onyx and Resolute Integrity are trademarks or registered trademarks of the Medtronic Group of Companies. Xience and Xience Skypoint are trademarks or registered trademarks of the Abbott Group of Companies. Biofreedom and BioMatrix are trademarks or registered trademarks of the Biosensors International Group. Cre8 is a trademark or registered trademark of the Alvimedica Group of Companies. Nobori is a trademark or registered trademark of the Terumo Group of Companies. Cypher is a trademark or registered trademark of the Cardinal Health Group of Companies. Clinical data conducted with Orsiro, Orsiro Mission’s predecessor device can be used to illustrate Orsiro Mission clinical outcomes.